### Intended use

The Apo A1, A2 & B High Level Calibrator is designed to be used with the Wako Apolipoprotein A1-HA, A2-HA and B-HA tests for the determination of Apolipoprotein A-I, A-II and B in serum or plasma.

### Reagents and ingredients

Apo A1, A2 & B High Level Calibrator Containing human serum	CAL: 1 x for 1 mL Store at 2 - 10°C.
Apo A1, A2 & B High Level Calibrator Diluent Containing 0.09% Sodium Azide (NaN <sub>3</sub> )	DIL: 1 x 3 mL Store at 2 - 10°C (Do not freeze).

### Determination of assigned values

The assigned values are determined by turbidimetric immunoassay and based on IFCC Reference Preparation SP1-01 for Apolipoprotein A-I and IFCC Reference Preparation SP3-07 for Apolipoprotein B.

### Preparation of reagent

Accurately add 1.0 mL of Apo A1, A2 & B High Level Calibrator Diluent to a bottle of Apo A1, A2 & B High Level Calibrator and dissolve the whole content (stock solution). Dilute the stock solution with the Diluent according to the following table and prepare 5 solutions. The solutions are stable for 2 weeks at 2-10°C (Do not freeze).

Solution No.	1	2	3	4	5
Stock Solution (µL)	50	100	200	300	Stock Solution
Diluent (µL)	350	300	200	100	0
Apolipoprotein concentration (mg/dL)	Assigned values x 1/8	Assigned values x 1/4	Assigned values x 2/4	Assigned values x 3/4	Assigned values

### Warnings and precautions

- For *in vitro* Diagnostic use.
- Only for professional use.
- Not to be used internally in humans or animals.
- Use Apo A1, A2 & B High Level Calibrator according to the instructions herein. Read instruction sheets of Wako Apolipoprotein A1-HA, A2-HA and B-HA.
- Allow the reagents to equilibrate to room temperature before use.
- Store the reagents under the specified conditions. Do not use the reagents past the expiration date stated on each container label.
- Do not use the reagents described above in any procedures other than those described herein.
- If the reagents come in contact with mouth, eye or skin, wash off immediately with plenty of water. Consult a physician if necessary.
- The Apo A1, A2 & B High Level Calibrator is prepared from human sera which were found to be negative for HBsAg, anti-HIV1 & 2 antibody and anti-HCV antibody. Because no test can offer complete assurance that products derived from human sera will not transmit infectious agents, it is recommended that the Calibrator should be handled with the same precautions used for patient specimens.
- All blood derivatives should be considered potentially infectious.
- Be careful not to cut yourself with the aluminum cap when remove it from the vial.
- The Apo A1, A2 & B High Level Calibrator Diluent contains sodium azide as a stabilizer. Sodium azide may react with lead or copper plumbing to form explosive compounds. Even though the reagents contain minute quantities of sodium azide, drains should be flushed well with a large amount of water, when discarding the reagents.
- When discarding the reagents, dispose of them according to local or national regulations.

### Ordering information

Code No.	Product	Package
415-77202	Apo A1, A2 & B High Level Calibrator	Set CAL: 1 x for 1 mL DIL: 1 x 3 mL

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Wako Chemicals GmbH