Intended use

The CH50 Calibrator is intended for joint usage with Wako Autokit CH50 for the determination of total complement activity in serum.

Packaging

CAL 1	1 vial x for 0.5 mL
CAL 2	1 vial x for 0.5 mL
CAL 3	1 vial x for 0.5 mL
CAL 4	1 vial x for 0.5 mL
CAL 5	1 vial x for 0.5 mL

Contain human serum.

Calibrator preparation

For reconstitution of contents, add precisely 0.5 mL distilled or deionized water to each Calibrator vial. The reconstituted solution must be stored on ice and used up (or eventually discarded) within a day.

When you get ready to reconstitute, please follow this procedure:

1. Remove vials from freezer.
2. Remove the cap and stopper and gently tap down all lyophilised material to the bottom of the vial.
3. Immediately add the appropriate amount of cold (2-8 °C) distilled water directly to the lyophilised material.
4. Replace the stopper securely.
5. Making sure that all the lyophilised material is covered with fluid, allow the vials to stand at room temperature for 5 minutes with occasional, gentle shaking.
6. When all the material has been reconstituted at the bottom of the vial, turn the vial on its side and allow the reconstituted material to "roll around" the walls of the vial by rotating the vial slowly.
7. Invert the vial and allow the fluid to cover the stopper for 20-30 seconds with gentle shaking.
8. Return the vial to the upright position and remove the stopper to drain down.
9. Replace the stopper and place the vial in an ice bath.
10. Allow the vial to remain on ice for 20 minutes more with occasional, gentle shaking.
11. Use the material as directed. **DO NOT REFREEZE THESE MATERIALS!**

Assigned values and their determination

Value assigned:	see bottle label
Determination:	according to Mayer ¹

Storage

Store at - 10°C or below.

Warnings and precautions

- For *In vitro* diagnostic use.
- For professional use.
- Not to be used internally in humans or animals.
- Do not use this Calibrator for any other use than described herein.
- The application of this test is reserved for professional use only. Please refer to applicable national and regional regulation and legislation also.
- Clinical diagnosis must be determined with clinical symptoms and other test results by a physician.
- The CH50 Calibrator has been prepared from human serum that was tested for HBsAg, anti-HIV antibodies and anti-HCV antibodies and found negative. As no known procedure can exclude with certainty the risk of infection from such blood-derived product, the Calibrator must be handled with the same precaution as human patient samples.
- When discarding the reagents, dispose of them according to local or national regulations.

References

1. Mayer MM. Complement and complement fixation. In: Kabat EA, Mayer MM, eds. Experimental immunochemistry, 2nd ed. Springfield, IL: Charles C Thomas, 1967: 133-240.

Ordering information

Code No.	Product	Package
997-43801	CH50 Calibrator	CAL: 5 conc. x for 0.5 mL

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Wako Chemicals GmbH