

METHOD: Turbidimetric Immunoassay
for Determination of RF in Serum

Ordering information

Reagents	Package	Code-No.
RF-HA II Reagent 1	R1: 2 x 50 mL	419-70757
Reagent 2	R2: 2 x 15 mL	
Multi-point calibration		
RF-TIA Calibrator Set	CAL: 5 conc. x for 1 mL	419-77902

Hitachi 7600/ Roche MODULAR

Reagent	RF-HA II
ANALYZE	
CH TEST/ TYPE	RF / SERUM
ASSAY	2POINT END - 10
POINT	16 - 34 - 0 - 0
WAVELENGTH(SUB/MAIN)	700 / 340
SAMPLE VOL. (NORMAL)	11.0 - 0.0 - 0
(DEC.)	10.0 - 0.0 - 0
(INC.)	22.0 - 0.0 - 0
DILUENT	H2O - 0
REAGENT VOL. R1	180 - 0 - () - 0
R2	0 - 0 - () - 0
R3	54 - 0 - () - 0
R4	0 - 0 - () - 0
ABS LIMIT	DECREASE
PROZONE LIMIT	0 0 LOWER
CELL DET.	DET. 1
CALIBRATION	
CALIB. TYPE	SPLINE
POINT / SPAN	6 / 6
WEIGHT	0
UNIT	IU/mL
AUTO CALIBRATION	0
SD LIMIT	999.9
DUPLICATE LIMIT	10% / 500 Abs
SENSITIVITY LIMIT	-99999 - 99999
S1 ABS LIMIT	-32000 / 32000
CALIBRATOR	CONC. POS. VOL. PREDIL
1	0 *1 (saline) 11.0 0.0
2	*1 *2 11.0 0.0
3	*1 *2 11.0 0.0
4	*1 *2 11.0 0.0
5	*1 *2 11.0 0.0
6	*1 *2 11.0 0.0
K-FACTOR	

*1: Input the concentration of the calibrator

0110D3/vk

*2: Input the position of the calibrator