Read this instruction sheet thoroughly before use!

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270-77000
0318D3

Intended use

The Wako NEFA Standard is designed to be used with Wako NEFA-HR(2) reagent for the quantitative determination of the free fatty acids (FFA) in serum.

Reagents and ingredients

CAL: 2 x 10 mL.
Contains: Oleic acid, Stabilizer

Reagent preparation

The standard is supplied in a liquid form.

Assigned values and their determination

Assigned value: see bottle label.
Traceable to Wako internal standard.

Storage condition

Store at 2–10 °C.
Expiration date: see bottle labels or package.

Warnings and precautions

- For *in vitro* diagnostic use.
- The usage and application of this test is reserved for professional use only. Please refer to respective national and local regulations and legislation.
- Not to be used internally in humans or animals.
- Do not use reagents past the expiration date stated on each reagent container label.
- Do not use the calibrator for any other purpose than described herein.
- This product contains stabilizers. Sodium azide may react with copper or lead plumbing to form explosive compounds. Even though this product contains minute quantities of sodium azide, drains should be flushed well with a large amount of water when discarding the solution.
- If the standard comes into contact with mouth, eyes or skin, wash off immediately with a large amount of water. Consult a physician if necessary.
- When discarding this product, dispose of it according to local or national regulations.

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